

SES 2010 Touchscreen Non-vacuum Autoclave

The Little Sister SES 2010 offers over 50 years of tried and trusted performance.

The latest generation of one of the world's best selling non-vacuum autoclaves includes a colour touchscreen, modernising the unit and bringing it to the same high specification as all our other autoclaves.


As well as integrating the easy to use touchscreen, the unit also provides:

- **Better Cycle Control**
Enables the user to select a cycle, with or without drying, at any time.
- **Improved Cycle Monitoring**
Easy to read, at a glance touchscreen display lets the user know exactly what stage the cycle is at.
- **Simple Cycle Approval**
User is able to acknowledge every cycle, helping to ensure full cycle validation in accordance with guidance. With the addition of the USB port, cycle records can be downloaded at a later date.



Contact Eschmann for more information
+44 (0) 1903 875 787
or head over to our website
www.eschmann.co.uk

Key Features

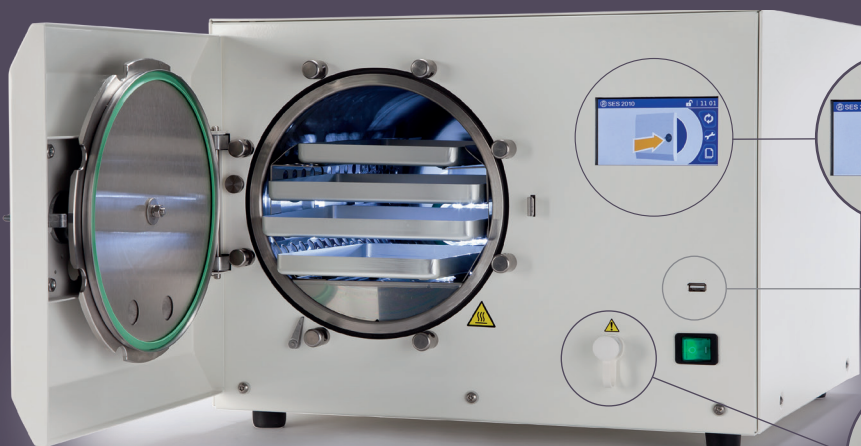

Colour Touchscreen

Easy to read and intuitive display modernises the unit, bringing it to the same high specification as all our other autoclaves.

USB Cycle Record Download

Enables the user to be able to easily download cycle records.

Detachable Drain Tube

Located on the front of the autoclave for easier drainage of the water reservoir.

Note: Trays shown for illustrative purposes only. Trays are sold separately.

Technical Specification

Autoclave Type:	N Type (non-vacuum)
Load Type:	Suitable for solid unwrapped instruments
Chamber Capacity:	11 Litres
Cycle Options:	134°C cycle with or without drying
Cycle Times:	From 12 mins (depending on load)
Cycle Data Management:	Option to connect wireless AutoLog cycle data manager and e-Logbook or thermal paper printer
Maximum Load Capacity:	4.5 kg. Furniture rack configured for... <ul style="list-style-type: none"> • 6 x 'examination' instrument trays or HFIT10 cassettes <li style="text-align: center;"><i>or</i> • 2 x 'standard' instrument trays or HFIT20 cassettes and 2 x 'examination' instrument trays or HFIT10 cassettes (instrument trays & HFIT cassettes sold separately)
Water Capacity:	2.5 Litres
Chamber Dimensions:	200 mm (diameter) x 350 mm (depth)
External Dimensions:	468 mm (d) x 460 mm (w) x 360 mm (h)
Shipping Dimensions:	605 mm (d) x 655 mm (w) x 468 mm (h)
Weight (empty):	26.5 kg
Gross weight (incl. shipping carton):	31.5 kg
Power Supply:	230 V (±10%) or 220 V (-6%, +15%)
Fuse Rating (plug):	13 A

The details in this brochure are correct at time of going to press and supersede details previously published elsewhere. Information, technical specifications and statements made within this guide may be subject to change without prior notice.

Ordering Information

Product No.	Product Description
8720100	Little Sister SES 2010 Non Vacuum Autoclave
8701010	Little Sister Autolog incl. E-Logbook & cycle data manager
8721367	Thermal Printer

All Little Sister SES 2010 autoclaves include: Mains Lead, User Manual, Wall Chart, USB Memory Stick with Cycle Data Download Software and Manuals, Reservoir Drain Tube, Drain Plug, Tray Carrier, Levelling Kit, Tray Lifter, Drip Tray and Reservoir Lid



ESCHMANN

Eschmann Technologies Ltd
 Eschmann House, 15 Peter Road, Lancing,
 West Sussex, BN15 8TJ, United Kingdom
T +44 (0) 1903 875 787
E enquiries@eschmann.co.uk
W www.eschmann.co.uk

